

KUNDE HEALTHCARE SERVICES

Bariatric Conversion Kit – 77111

HALO

ASSISTED LIVING CENTERS®

ITEMS INCLUDED IN THIS BOX:

2 EACH – 15 INCH Halo Width Extenders

2 EACH – BOLTS


2 EACH - NUTS

2 EACH – LOCK WASHHERS

2 EACH – WASHERS

INSTALLATION:

- Slide each 15 inch Halo Width Extender with the holes one inch apart over each of the brackets of the halo safety ring.
- Adjust the width extenders over the Halo Safety Ring brackets to the desired bed width.
- Place each bolt and washer over the top hole of the extender of the hole that will secure the width desired.
- Secure the bolt with the washer, lock washer and nut on the bottom side of the width extender bracket and the halo safety ring bracket.
- Continue installing the Halo Safety Ring by using the halo safety ring direction provided.

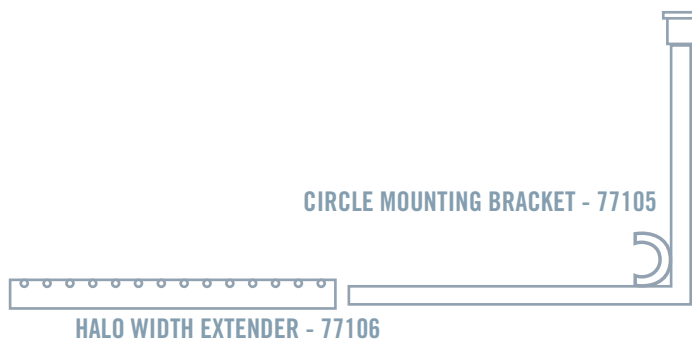

Note: It is recommended that the use of a Halo double sided mount (77102) be used when applying the bariatric conversion kit. Please note: If you purchase halo safety double mount for for the Biariatic Bed, Kunde Health Care Services will install the width extenders with hardware on the mounting brackets of the Halo Safety Ring double mount and will ship together ready for installation.

www.kundehealthcareservices.com

1261 Shoal Ridge Road / Oconomowoc, WI 53066
Telephone 262-370-4556 / Fax 262-567-8646
Email: mkunde7@sbcglobal.net