

KUNDE HEALTHCARE SERVICES

PRESENTS THE HALO BACK FOUR ALL

HALO
SAFETY RING



77119 - Halo Back Four All with Fusion Cover

77120 - Halo Back Four All with Wicking Cover

THE HALO BACK FOUR ALL

NEW WHEEL CHAIR BACK TECHNOLOGY WHICH...

1. Allows therapists to customize back fittings for Kyphosis, Scoliosis, and other Spinal Curvatures while healing ulcers in the Thoracic and Coccyx regions.
2. Increases seat depth up to five inches for Bariatric residents without jeopardizing the integrity of proper positioning and upper body support in the Bariatric resident.
3. Redistributes redundant tissue in residents, encouraging proper seating and positioning. It adjusts to weight gain or loss without having to purchase a new chair.
4. Heals both Thoracic and Coccyx ulcerations while the resident is still in their chair and not in their bed as it had been typically performed in the past.

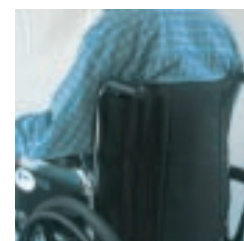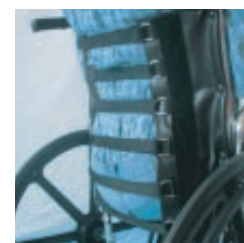

INSTALLATION REQUIREMENTS

1. Remove existing upholstery Back.
2. Insert one top screw on each side of back canes.
3. Wrap left and right Velcro connectors around each back cane.
4. Adjust Halo Strapping System for Residents Comfort and Safety.
5. Finalize Installation by Applying One Foam Comfort Cover.

SIZING

Fits all wheel chairs from 15"–21" wide.
All Sizes 21"–28" are price out solely
based on additional material used.

www.kundehealthcareservices.com

1261 Shoal Ridge Road / Oconomowoc, WI 53066
Telephone 262-370-4556 / Fax 262-567-8646
Email: mkunde7@sbcglobal.net