

KUNDE HEALTHCARE SERVICES

PRESENTS THE HALO SAFETY RING

HALO
SAFETY RING

77116 - LIMB PROTECTOR

SIZING- One Size Fits All (Universal)

FEATURES & BENEFITS


- Developed for Dementia and Alzheimer's patients, the Halo Safety Ring Limb Protector secures and prevents the upper limb from penetrating any one part of the Halo Circle, while allowing the residents to ergonomically grip the device.
- The Halo Safety Ring Limb Protector is transparent, therefore eliminating any additional isolation for the Dementia and Alzheimer residents.
- The Limb Protector can be hand washed and air dried.

HISTORY

The design for this device was a response to a request from the State of Pennsylvania surveyors who were concerned that the Alzheimer's and Dementia residents were not extended the same rights and capability for use of the Halo Safety Ring. State surveyors were concerned that Alzheimer's and Dementia patients' limbs might become caught or injured while using the Halo Safety Ring. The Halo Safety Ring Limb Protector was developed to alleviate that afore mentioned concern. Kunde Healthcare Services, Inc wishes to express our thanks to the State of Pennsylvania's surveyors for allowing us to create a device which will benefit long term treatment facilities throughout the United States!

INSTALLATION REQUIREMENTS

1. Remove Halo Safety Ring, Limb Protector from plastic bag.
2. Unzip both sides of the Limb Protectors zippers.
3. Slide the Halo Safety Ring, Limb Protector over the Halo Circle.
4. Finally zip both sides of the Halo Safety Ring, Limb Protector together stopping at the center of each side of the Halo Safety Ring, Rubber Tube Separator.

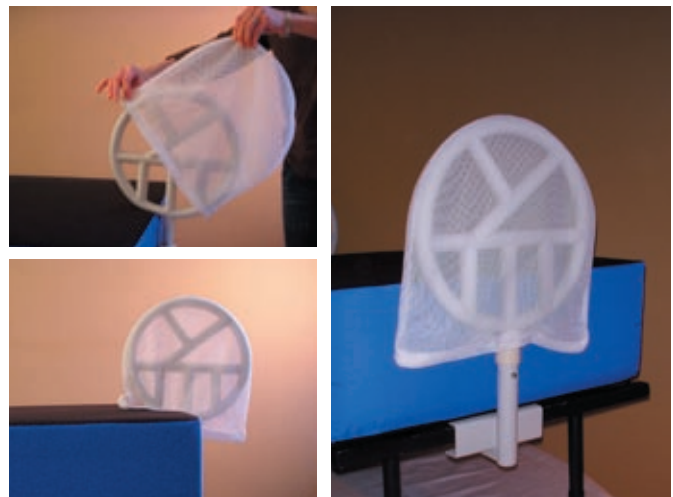

www.kundehealthcareservices.com

1261 Shoal Ridge Road / Oconomowoc, WI 53066
Telephone 262-370-4556 / Fax 262-567-8646
Email: mkunde7@sbcglobal.net