


PRESENTS THE HALO SAFETY RING

CIRCLE PAD PROTECTOR

77118 - Halo Safety Ring, Circle Pad Protector

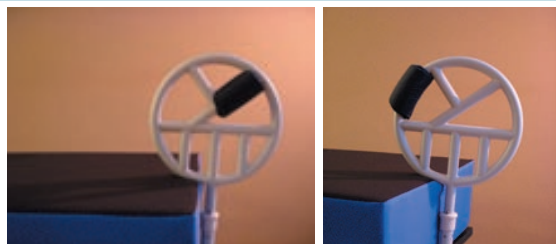

FEATURES & BENEFITS


- The Halo Safety Ring Circle Pad Protector prevents bruising from any medical disorder that might cause the skin to become thin, frail, or bruise (specifically seizures or bed thrashing).
- The Pad Protector also prevents any accidental environmental injury in the resident's rooms due to a fall. Seizure and spastic residents that need padding for any type of protection due to their disorder would benefit from the Halo Safety Ring Circle Pad Protectors. This accessory will also promote comfort, safety, and a better sleeping environment for individuals.

INSTALLATION REQUIREMENTS

1. Halo Safety Ring Circle Pad Protector is purchased as a single unit and is packaged one per poly bag. Remove Halo Circle Pad Protector.
2. Remove outside cloth material with Velcro closure connectors.
3. Place open slit of rubber noodle around determined area of Halo Circle for best results per resident application.
4. Rewrap cloth material with Velcro closure over rubber noodle and secure.
5. Redo steps 2, 3, and 4 for each Halo Circle Pad Protector that will be used on each Halo Safety Ring.
6. A maximum of three Halo Circle Pad Protectors on the top half of each Safety Ring is recommended.

U CHANNEL RUBBER SPACER

77115 - Halo Safety Ring, U Channel Rubber Spacer
SIZING- Fits all Halo Safety Ring U Channel Mounting Brackets

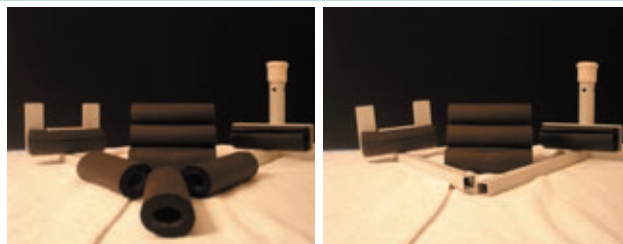

FEATURES & BENEFITS

- The Halo Safety Ring U Channel Rubber Spacer prevents the Halo Safety Ring Brackets from moving and sliding against the bed frame.
- The Halo Safety Ring U Channel Rubber Spacer allows for a tighter fitting turnbuckle, making for a tighter fitting Halo Safety Ring mounting bracket system.
- The Halo Safety Ring U Channel Rubber Spacers help to hold up the Halo Mounting Brackets while they are connected to the turnbuckle.
- The U Channel Rubber Spacers protect the sides of the bed frame from scratches and damaging the life of the bed.

INSTALLATION REQUIREMENTS

1. Remove Rubber Spacer from plastic bag.
2. Open Slit on U Channel Rubber Spacer and position on the frame of the bed where the Halo Safety Ring Mounting Brackets will over-lap the U Channel Rubber Spacer and secure itself.
3. Finish the installation according to the instructions or the interactive video on the Kunde web page at www.kundehealthcareservices.com