



INSTALLATION INSTRUCTIONS FOR SINGLE SIDED BED MOUNT

**PRODUCT NUMBER 77101**

**HALO**  
SAFETY RING

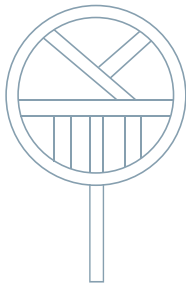

Item No. 77104

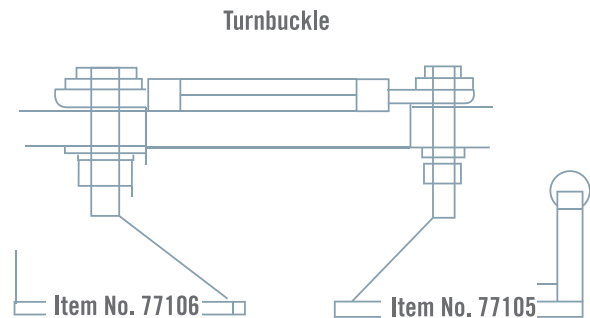

## ITEMS INCLUDED IN THIS BOX:

- 1 Halo Safety Ring – Item No. 77104
- 1 Halo Safety Ring Mount Bracket - Item No. 77105
- 1 Halo Safety Ring Mattress Stay – Item No. 77106
- 1 Halo Safety Ring Hardware Kit – Item No. 77109

## ITEMS INCLUDED IN THE HARDWARE KIT - Item No. 77109

- 1 8 ½" Tension Bar
- 1 Turnbuckle
- 2 5/16" Zinc Plated Bolts
- 4 5/16" Zinc Plated Washers
- 4 5/16" Zinc Lock Washers
- 2 5/16" Zinc Nuts

## TOOLS NEEDED FOR ASSEMBLY:

- 9/16" Open End Wrench (for use on the turnbuckle)
- 1/2" Closed End Wrench (for use on the bolt head)
- 1/2" Extension Socket with Ratchet Wrench (for use on the nuts)

**Kunde Healthcare Services**

1261 Shoal Ridge Road / Oconomowoc, WI 53066  
Telephone 262-370-4556 / Fax 262-567-8646  
Email: [mkunde7@sbcglobal.net](mailto:mkunde7@sbcglobal.net)

# INSTALLATION STEPS:

1. To begin, remove all original mounting hardware from the bed frame. Raise the head portion of the bed to the highest position.
2. Attach Halo Safety Ring Mount Bracket (77105) 2" above the hinge that raises the head portion of the bed frame. Attach Halo Safety Ring Mattress Stay (77106) on the opposite side of the bed frame, 2" above the hinge.
3. Connect the Halo Safety Ring Mount Bracket and the Halo Safety Ring Mattress Stay to the turnbuckle and tension bar using the enclosed bolts, washers, lock washers and nuts.
4. Loosen the turnbuckle 1" on both sides.
6. Check the alignment of both the Halo Safety Ring Mount Bracket and the Halo Safety Ring Mattress Stay to make certain both are positioned in the same place on either side of the mattress frame. Straighten if necessary. Tighten the turnbuckle with the 9/16" open-end wrench until the turnbuckle begins to rise up to the bottom portion of the frame. Eliminate all space between the mattress frame and the Halo Safety Ring Mount Bracket by pushing down from the top side of the frame.
7. Once secure, slide the Halo Safety Ring (77104) into the Halo Safety Ring Mount Bracket by pinching the double locking pins on the bottom bar of the Halo Safety Ring. Once inserted, the locking pins will pop out securing the Halo Safety Ring in position.
8. Once assembled and locked into place, adjust the tension bar if necessary to prevent any movement from the Halo Safety Ring Mount Bracket or the Mattress Stay.

Your Halo Safety Ring is now installed and ready for resident use.

Patent Pending. Halo Safety Ring is a registered trademark of Kunde Healthcare Services, Inc.

The Halo Safety Ring comes with a lifetime guarantee. For service or to order replacement parts call 262-370-4556.

**Kunde Healthcare Services**

1261 Shoal Ridge Road / Oconomowoc, WI 53066  
Telephone 262-370-4556 / Fax 262-567-8646  
Email: [mkunde7@sbcglobal.net](mailto:mkunde7@sbcglobal.net)